

RECOMMENDED PROTECTIVE BARRIER COVERING VASCULAR ACCESS FOR INFUSION THERAPY

BENEFITS

- + Waterproof, permeability tested
- + Self-adhering (does not stick to skin or hair)
- + Transparent (allows visualization of site)
- + Allows patients with IVs or PICCs to shower while keeping site dry and secure
- + Easy to use by any patient care giver

HOW TO USE

- + Apply SealSkin Wrap 2 to 3 inches above and below site to ensure a watertight seal
- + While wrapping, keep SealSkin taut to provide better control and coverage
- + Use scissors or your thumb to tear the wrap off at the desired length and seal the end in place with your hand

TO REMOVE

- + Use scissors, unravel or tear wrap
- + Always assess the integrity of the site after removal



Hospital Tested and Approved
Passed Bio-Compatibility Testing
FDA Registered
Veteran Owned/WBENC Certified

